

Quick User Guide to Roger touch and Roger X02 Ear level receivers



Components

Roger Touch transmitter

Roger receiver



STEP 1: Switch on microphone

Press and hold the on/off button until indicator light GREEN.



A STEP 2: Fitting and switching on pupils receiver

Connect receivers to the hearing aid via audio shoe (three pin euro plug).
Please note on only Type 02 receivers work with the touch.



STEP 3: Pairing transmitter with receiver

Pair transmitter with each receiver, this should only have to be done once. Hold transmitter within a 15cm in range of the receiver and select CONNECT.



STEP 4: Increasing the volume

The volume can be increased/ reduced on each receiver via the menu. Scroll across to SETTINGS, hold close to the receiver your wishing to adjust, Select RECEIVER SETTINGS and select EASY GAIN. Use the +/- controls to adjust were required.





STEP 5: Charging

Plug transmitter charger in to the mains using the charger provided and plug in to the base of the transmitter until the charging symbol ⚡ appears.



Using transmitter

The transmitter can be used for a teacher or in a group situation (on table). This will operate automatically. For example when placed on a table the microphone will go in to conference mode.

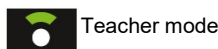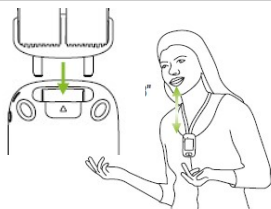

Muting the microphone

To mute the system press the MUTE button located at the base of the transmitter. Red is muted and Green is unmuted.



Connected to audio source

To link and audio source to the transmitter use a 3.5mm audio jack lead. Plug in to headphone socket on device and then plug in to base of transmitter

