

## Rebroadcasting from Frontrow JUNO to Roger Inspiro



Prioryfield House, 20 Canon Street, Taunton, Somerset,  
TA1 1SW, 01823 323363 / 07941015282,  
info@robertsaudiosolutions.co.uk,  
www.robertsaudiosolutions.co.uk



**Audio out**

**Audio input**

- Using a 3.5mm jack to jack lead connect to the AUDIO OUTPUT from the JUNO,
- With the other end of the lead connect the Roger Inspiro transmitter using the AUDIO LEAD in to the base of the transmitter,
- Switch on the Roger system and UNMUTE the Juno microphone,
- Whilst talking in to the JUNO microphone increase the AUDIO OUTPUT control to around volume 8,
- Ensure the hearing instrument user can hear the JUNO microphone but also the environmental sounds via the hearing aid.

### Roger Touch



**Audio input**

### JUNO Teacher Microphone



### JUNO Student Microphone

